

# SERVICE BULLETIN

---

Carestream Health, Inc., Rochester, NY 14608

**AUGUST 2007**

## ***KODAK Point-of-Care CR 120/140/260 SYSTEM***

**with Auto-loop Modifications**

**Service Code(s): 3607, 3605 and 3606**

### **New Cleaning Tray and Plate**

#### **Background**

Recent model of PoC scanners are shipped with a mechanical design change called "auto-loop".

This new design replaces the plate-guide-loop (Mylar) which was causing scratches on the plates with a new solenoid.

Following to this change, new cleaning tray and cleaning plate are released in order to be used with "auto-loop" systems.

#### **How to identify "auto-loop" systems**

Looking at the key assembly (above the front tray) allows identifying whether system has new "auto-loop" design or not.

Existence of the hole marked in red on the key assembly (above the tray) indicates this system has new "auto-loop" design:

